


# EFR KIT


Easy Implant Removal KIT


**OSSTEM<sup>®</sup>**  
IMPLANT

- Bone loss-less fixture removal
- Quick & Simple removal by two stage
- Above 80% removal success

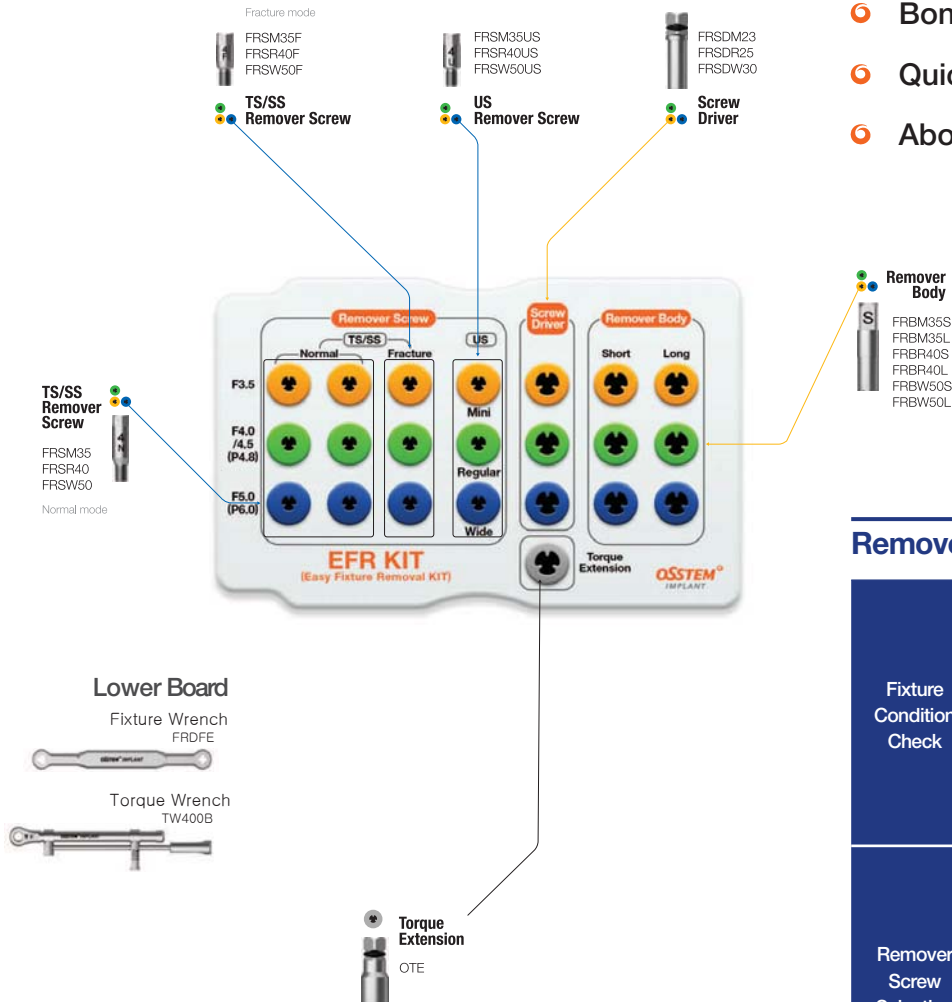


## Remover Screw Selection Guide

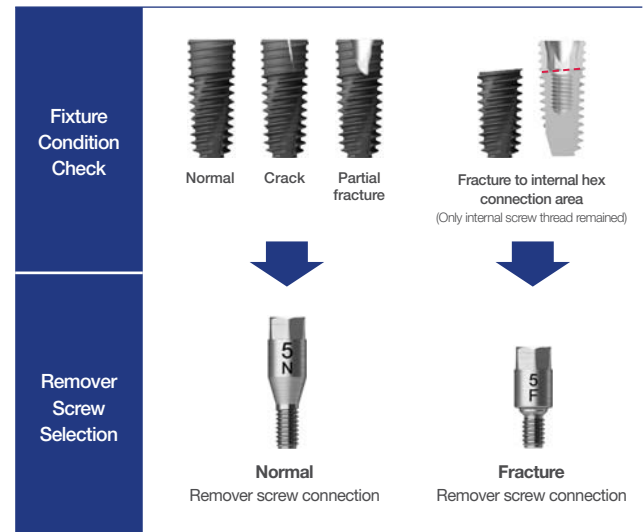

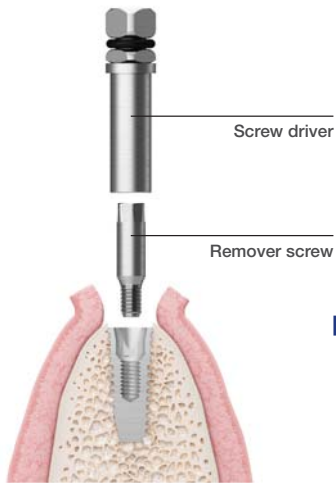


1. Select remover screw  
(Refer to the remover screw selecting guide)

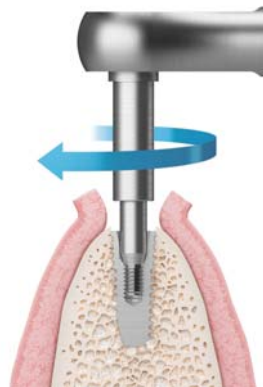


2. Clockwise tightening by torque wrench  
through the screw driver  
(mini 80Ncm, regular 100Ncm)



3. Complete the removal screw  
installation into the fixture

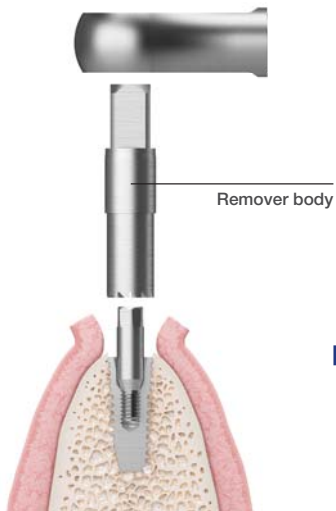

4. Removal body selection

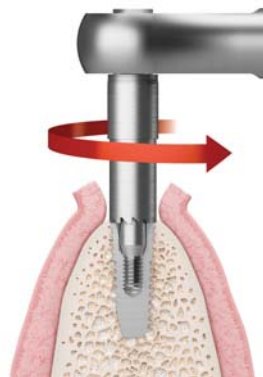

5. Anti-clockwise screwing by torque wrench  
through the removal body  
(Maximum 400Ncm engagement possible)



6. Bone loss-less fixture removal by  
strong adhesion property between fixture & removal body